



A new level of care and access

The Clinician Access Portal

Physician Quick Reference



8522 W. CAPITOL DR.
MILWAUKEE, WI 53222

P. 414.847.1800 • F. 414.847.1820

10500 W. LOOMIS RD. SUITE 120
FRANKLIN, WI 53132

P. 414.529.3100 • F. 414.529.3102

WWW.PDI-RADIOLOGY.COM



The Clinician Access Portal (CAP):

The Clinician Access Portal is a dynamic new image-enabled Web portal that provides secure, convenient access to current and historical patient reports and images. It also relays patient and appointment status and enables emergency/referral access for hospital/clinic collaboration, or in an emergency situation.

This quick reference guide is designed to help you log on and use the CAP.

Logging On:

To log on to the Clinician Access Portal, open your internet browser and enter www.pdi-imaging.com in the address bar. Click on the orange box entitled "Clinician Access Portal." When the login screen appears, enter your user name and password and click the "Log In" button to launch the CAP.

Forgot Your Password?

If you cannot remember your password, click the "Forgot Password" link on the login screen. Enter your email address and click the Submit button. A temporary password will be sent to your email account, which you will use to log on and then change after the first use.

Changing Your Password?

You can change your password at any time while logged on to your account through your Profile page. Click the "Change Password" link and enter your old and a new password. Click the "Submit" button and use this password the next time you log on.

Main Features

The CAP provides the following features to better serve your practice:

Dashboard:

Real-time patient information displayed in snapshots for easy review and access. Navigation to the information you need is managed through multiple screens that link to their appropriate information regarding your patient's exam status, reports, images, appointments and much more.

Study Display and Visualization Tool Set:

Access current and prior studies for comparison with an extensive tool set for study manipulation. Additionally, you can load multiple series, perform measurements, utilize window level presets and default series layouts by modality.

Pre-Registration:

Reduce the need for phone calls and increase accuracy in the ordering process. Enter insurance information such as pre-certification and insurance verification to further streamline the order entry process. Submitted information appears on the PDI Scheduling page. Please continue to fax all orders to PDI.

Patient Search:

Locate and view reports for all of your patients.





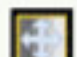


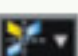








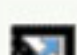


Emergency/Referral Access:

Securely access patient information for cases of emergency or referral. Requests made via this page are logged.


Report Printing:

Display and print reports with key images, exactly as they are displayed at PDI. Bulk group and print copies of patient reports from the Recent Reports page with a single click.

Study Display and Visualization Tool Set:

- | | |
|--|--|
|  Apply parameters to one image or to all images in a series. |  Compare images of different planes by turning on the Localizer Lines tool. |
|  Select a predefined number of Viewers and a predefined image layout in the Viewer. |  Turn off Localizer lines in the image by clicking the Localizer Lines tool again. |
|  Adjust the window level width or height using the mouse. |  Magnify an area of interest. |
|  Zoom in on an area of the image. |  Flip images vertically or horizontally. |
|  Drag an image to a new position in the viewport. |  Reset the image display parameters to the setting applied when the study was first loaded. |
|  Rotate images to the right. |  Display overlay text on images |
|  Rotate images to the left. |  Select the previous series within the selected study. |
|  Change window settings automatically to presets by modality. |  Select the next series within the selected study. |
|  Invert the image grayscale for a negative image display. |  Select the previous image in the series. |
|  Go to full browser mode and suppress the left panel. Click the button again to return to normal mode. |  Select the next image in the series. |



 The Annotate tool is a multifunction tool that enables you to make measurement and text annotations.

Line

Line

Click two points on the image to create a line.

ROI with a straight line between the start and end points.

Rectangle

Rectangle

Click and drag the mouse pointer to draw a rectangle over the desired region.

Angle

Click three points on the image to display the angle's measurement.

Ellipse

Ellipse

Click and drag the mouse pointer to draw an ellipse over the desired region.

Cobb Angle

Create two lines to extrapolate the point where the lines intersect to display the Cobb angle.

ROI

ROI

Click and drag the mouse pointer to draw a free-hand region of interest (ROI). When you release the mouse button, the application completes the

Text

From the Text menu, click Add. Click in the image and enter text.

Angle

Cobb Angle

Text

Edit

Delete

Remove All

10 Reasons to Use the Clinician Access Portal

1. Lets you and your staff access reports and images quickly and effortlessly, with "real time" information
2. Lets you access your patients reports and images from any computer, at any location—without the need for additional hardware, software, viewer plug-ins and without concern for IT firewalls or security policies
3. Makes your patients happier by reducing the time they wait for results
4. Compatible with MAC and Windows
5. Makes the workflow of your front office more efficient (and your staff happier)
6. Allows a colleague to view the report and images almost instantly in an emergency or as a secondary read
7. Gives your practice a high-tech image
8. Saves time by eliminating lost films and reports, re-faxing and courier service
9. Provides secure HIPAA-compliant access to patient information
10. Ability to show patients their images in your office



Accessible & Convenient

Patients of all ages and same day appointments are always welcome. PDI has two locations to serve you.

Most insurance plans are accepted. A complete list can be found on our website at www.pdi-imaging.com. A low self pay fee schedule is also available.

Milwaukee

8522 W. Capitol Drive
Milwaukee, WI 53222

phone 414.847.1800
fax 414.847.1820

Hours: 8 am - 7 pm Monday - Friday
9 am - 1 pm Saturday

Franklin

Located in OneSource Medical Center
10500 W. Loomis Road, Suite 120
Franklin, WI 53132

phone 414.529.3100
fax 414.529.3102

Hours: 8 am - 7 pm Monday - Friday



Milwaukee & Franklin Locations

